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Kirsch kern-Kissen

D	Kirsch kern-Kissen – Gebrauchsanweisung	2
GB	Cherry stone pillow – Instructions for use.....	7
F	Coussin de noyaux de cerise – Instructions d'utilisation.....	12
I	Cuscino con noccioli di ciliegia – Istruzioni per l'uso	17
NL	Kersenpitkussen – Gebruiksaanwijzing.....	22
TR	Kiraz Çekirdeği Yastığı – Kullanım Kılavuzu.....	27
RU	Подушка с наполнителем из вишнёвых косточек – Инструкция по применению	31

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Artikelnummer

Chargennummer

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Cherry stone pillow – Instructions for use

Dear Customer,

Below you will find important information about this product. Please read this carefully, in particular the safety advice, as well as the advice on how to use and care for the product.

Please follow the instructions and retain the instruction sheet and the packaging!

LOT See packaging!

Made in Germany

1. Application

These instructions for use are applicable for the following products:

- REF** 100-00 Cherry stone pillow 19x19 cm
- 110-00 Cherry stone pillow 22x24 cm
- 112-00 Cherry stone pillow 16x16 cm
- 1100-00 Cherry stone pillow 43x20 cm

2. Contents

Cherry stone pillow, Instructions for Use

3. Purpose and Intended Use

The cherry stone pillow can be applied in case of a generally feeling cold, and especially for easing pains in the context of a heat and cold treatment.

4. Areas of Application

The cherry stone pillow is exclusively suitable as a warmer for the following complaints:

- Stomach ache/Stomach cramps
- Constipation/Flatulence
- Cold
- Chronic bronchitis
- Chronic rheumatic complaints
- Muscle pains/stiffness
- Neck tensions
- Back pain
- Menstruation complaints
- Generally warming cold body parts

The cherry stone pillow is suitable only for the following applications as cold insert:

- Acute rheumatic complaints
- Lumbago/sciatica
- Headaches/migraine
- Bruises, strains, sprains
- Joint inflammations and stiffening
- Chronic irritation in ligaments, muscles or tendons
- Insect bites

5. Usage instructions

Heating options:

Cherry stone pillow 19x19 cm Cherry stone pillow 16x16 cm	Cherry stone pillow 22x24 cm Cherry stone pillow 43x20 cm
<ul style="list-style-type: none">• in the microwave at max. 300 watt for max. 1 minute (Follow the instructions of the microwave manufacturer!)	<ul style="list-style-type: none">• in the microwave at max. 300 watt for max. 2 minutes (Follow the instructions of the microwave manufacturer!)
<ul style="list-style-type: none">• in an oven that is not preheated, for max. 5 minutes at a max. temperature of 110 °C	
<ul style="list-style-type: none">• place on a hand-warm radiator or tiled stove for a longer period of time	

Shake the pillow immediately after heating to evenly distribute the heat in the pillow!

Moisture may appear on the surface of the pillow, when it is heated up for the first time, which will however disappear with repeated applications.

Cooling options:

To cool the cherry stone pillow, place it in a well-sealed foil cover and put it in an ice compartment or freezer for at least 30-60 minutes!

6. Attention! Important Safety Information!

Please observe the above directions on heating the pillow, as a longer heating time or higher temperatures may cause damage (risk of inflammation)! Always supervise the heating process of the pillow and do not heat the pillow when moist! Let the pillow cool down before reheating it!

Heat should not be applied in case of an acute inflammatory disease or an illness accompanied by a temperature, eczema, heart and artery disease, tumours, cancer and for persons with a reduced sensation of pain!

The application of cold should be avoided in case of undercooling, cold sensitivity, open wounds, arterial circulatory disorders, Raynaud syndrome, starting colds, bladder and renal diseases!

Please follow these additional instructions if the cherry stone pillow is used for babies/children:

Pay attention to intact seams! Loose cherry stones are a choking and suffocation hazard! Only use under direct supervision of an adult! After heating, check the temperature of the pillow by applying it to the crook of your arm to check the suitability for babies (max. 43 °C)! If the product is used for babies under 1 year of age, the child's temperature should be checked regularly during the application to avoid overheating or undercooling!

7. Care Advice and Storage

Do not wash the cherry stone pillow in order to avoid the formation of rot/mould! Store the product in a dry place and protect from humidity!

8. Information about Shelf Life and Disposal

It is recommended to replace the cherry stone pillow after a maximum of 4 years, if used normally. The pillow can be disposed of along with the household waste.

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