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Mikrowelle

Backofen

Organic Rape Seed Pillow – Instructions for Use

Dear Customer,

Thank you for buying this product. You have made a good choice by buying this natural product. Below we have compiled important product information for you. Please read this carefully, in particular the safety advice, as well as the advice on how to use and care for the product.

Please follow the instructions and retain the instruction sheet and the packaging!

LOT See Packaging!

1. Scope of Application

These instructions for use are applicable for the following products:

- REF** 611-00 Organic Rape Seed Pillow 19x19 cm
- 668-00 Organic Rape Seed Pillow 22x24 cm

2. Contents

Organic Rape Seed Pillow, Instructions for Use

3. Purpose and intended Use

The Organic Rape Seed Pillow can be used to apply heat to increase feelings of wellbeing and especially for easing the body's aches and pains as a form of heat therapy.

4. Areas of Application

The Organic Rape Seed Pillow is suitable as a warmer exclusively for the following areas of application:

- Stomach ache/Stomach cramps
- Constipation and flatulence
- Colds
- Muscle pains/stiffness
- Muscle/neck tensions
- Discomfort during menstruation
- Generally warming cold areas of the body

5. Usage and Heating Instructions

The following options are available to heat the pillow:

REF	611-00 Organic Rape Seed Pillow 19x19 cm	668-00 Organic Rape Seed Pillow 22x24 cm
In a non-preheated oven	at a max. temperature of 110 °C for max. 5 minutes	at a max. temperature of 110 °C for max. 6 minutes
In the microwave (Follow the instructions of the microwave manufacturer!)	at max. 300 watt for max. 1 minute	at max. 300 watt for max. 2 minutes
Place on a lukewarm radiator or tiled stove for a longer period of time		

Immediately after heating the pillow, distribute the heat evenly by shaking it! When heated for the first few times, you may find that there is slight moisture on the surface, but this will disappear after repeated use.

6. CAUTION! Important Safety Information

It is essential that you observe the specified directions on heating, as a longer heating time or higher temperatures may cause damage (risk of inflammation and burning)! Before heating the pillow in the microwave, check that heating grain products is permitted in the manufacturer's instructions for use. Always supervise the heating process of the pillow and do not heat the pillow when moist! Let the pillow cool down before reheating it! The product should not be used as a pillow or a seat under any circumstances.

Please follow these additional instructions if the Organic Rape Seed Pillow is used for babies/children:

After heating, test the temperature of the pillow to make sure that it is suitable for babies (max. 43 °C) by putting it in the crook of your elbow! Pay attention to intact seams! Loose rape seeds are a choking and suffocation hazard! Only use under direct supervision of an adult! If the seams have come open and loose seeds are leaking out, dispose of the product and the loose seeds immediately and make sure they are out of reach of children. If the product is used for babies under 12 months, the child's temperature should be checked regularly during the application to avoid overheating!

If the symptoms persist or worsen, please seek advice from a doctor. Heat should be avoided in case of an acute, chronic, inflammatory disease or an illness accompanied by a tempera-

ture, circulatory problems, fresh hematomas, bruises and open wounds as well as for individuals with a reduced sensation of pain! If there are existing skin conditions, heart disease, venous disease, diabetes or tumor disease, or if the user is pregnant please contact a doctor before use.

All serious incidents occurring in connection with this product must be reported to the manufacturer and the responsible authorities in the user's country of residence.

7. Care Advice and Storage

Do not wash the Organic Rape Seed Pillow in order to prevent the formation of rot/mould! Store the product in a dry place! Protect from moisture!

8. Information about Shelf Life and Disposal

We recommend replacing the Organic Rape Seed Pillow after a maximum of 4 years, if used normally. The product can be disposed of in household waste and recycled thermally without any problems. Please separate the individual packaging components and dispose of them in the relevant recycling system. Please be aware of regional disposal guidelines.

9. Material Composition

Material: 100% cotton (controlled organic cultivation)

Filling: Rape seeds (controlled organic cultivation)

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